RUCK® Massageliege mobil

RUCK® Mobile Massage Table

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Copyright

© 2023 HELLMUT RUCK GmbH. All rights reserved. This document may not be copied, edited, reproduced or translated without the prior written consent of HELLMUT RUCK GmbH.

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Table of contents

Declaration relating to these instructions for use	16
Abbreviations and symbols	16
Scope of delivery	17
Packaging	17
Assembly	17
Product description	18
Use of massage table and accessories	19
Maintenance	20
Care instructions I Cleaning recommendation	21
Accessories	23
Spare parts	24
Warranty	24
Technical specifications	25

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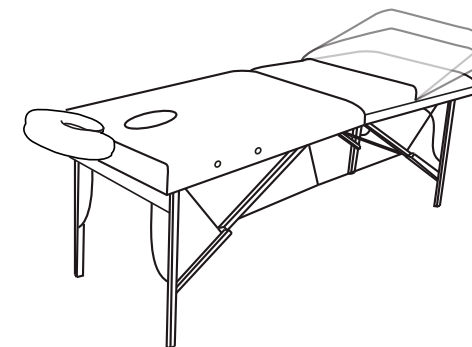
Declaration relating to these instructions for use

HELLMUT RUCK GmbH reserves all rights to modify and supplement these instructions for use. HELLMUT RUCK GmbH has taken great care to keep these instructions free of inaccuracies and omissions. Please contact us if you find any errors or inaccuracies. However, HELLMUT RUCK GmbH assumes no liability for mistakes in these instructions for use or consequential damage resulting from the provision of these instructions for use or their application when using the product.

These instructions for use must not be disseminated or supplemented without the express approval of HELLMUT RUCK GmbH.

The manufacturer cannot be made responsible for damage that is caused through use that deviates from the intended use or failure to comply with the safety instructions and warnings.

Abbreviations and symbols

This symbol identifies a potential point of danger for people or for the device itself. It must be taken note of.

**This symbol identifies useful advice.
Additional information about the product and how to use it is provided here.**

Observe the disposal notice

Scope of delivery

Check that the delivery is complete.

- 1 x massage table
- 1 x headrest
- 2 x armrests
- 1 x carrying bag with shoulder strap
- 1 set of instructions for use

Packaging

The packaging has been designed to protect the product while in transit. The packaging materials used have been selected with due regard for environmental issues and can be recycled.

Assembly

Lay the massage table on its side and loosen the fasteners. Unfold the massage table a little and remove the accessories.

Stand behind the massage table and grasp both carrying handles. With a forward movement, the legs of the massage table can easily be folded out completely.

Take the middle part of the massage table and place the table upright on the floor. Then press down on the massage table from above and make sure that it is completely flat. If the massage table points slightly upwards in the middle, check the safety ropes on the underside and loosen them if necessary.

The massage table is designed for a maximum weight of 200 kg. Exceeding this limit may result in injury to you and your clients.

Before use, make sure that all four feet are firmly on the floor. Only use the massage table on a flat surface.

Product description

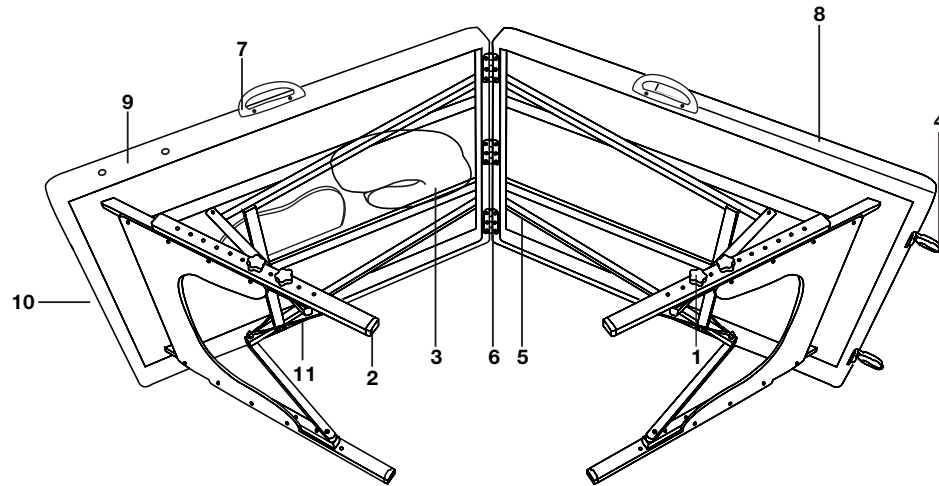
Figure 1

- | | |
|--------------------------------|-------------------------------------|
| 1. Knobs for height adjustment | 7. Carrying handle |
| 2. Rubber foot | 8. Adjustable backrest |
| 3. Headrest | 9. Insertion position for armrest |
| 4. Fasteners | 10. Insertion position for headrest |
| 5. Wooden struts | 11. Safety rope |
| 6. Hinges | |

Use of massage table and accessories

The scope of delivery of the massage table includes additional accessories. Please read the relevant instructions carefully before first use.

Headrest

The circular headrest provides optimal and comfortable positioning for your client's head and face. Both height and angle of inclination can be adjusted on the knob for the appropriate setting (Fig. 1 | 3).

Armrest

The armrests allow the massage table to be comfortably widened on both sides. This is particularly helpful with obese clients. The armrests, which are also padded, are simply inserted into the holes provided on both sides of the massage table (Fig. 1 | 9).

Height adjustability

The optimal height of the massage table depends on the person being treated as well as on the type and execution of the treatment. To adjust the height, remove the screws on the legs, slide them together or apart as desired and retighten the screws (Fig. 1 | 1).

Adjustment of the backrest cushion

Loosen the fastener of the back support. The angle of inclination can now be easily adjusted. Then press against the cushion and make sure it is locked in place. To fold the massage table, pull the back cushion all the way up and then fold it back into the completely flat position (Fig. 1 | 8).

Load-bearing capacity

Position your client in the middle of the massage table. The weight can thus be distributed evenly and the stability is increased. The working weight is composed of the client's weight and the maximum pressure applied during the treatment. The maximum load-bearing capacity of this massage table is 200 kg. Since lateral pressure puts more strain on the stability of the massage table, the lying surface should never be loaded on one side.

The massage table is designed for a maximum weight of 200 kg. Exceeding this limit may result in injury to you and your clients.

Storage

Attach the headrests and armrests to the underside of the massage table. Use the fastening options provided in each case (Fig. 1 | 10).

The carrying bag can be used for storage as well as for transport. In this way, the massage table is optimally protected. Store the massage table in a cool, dry place. Avoid direct sunlight, extreme temperatures (cold and heat) and humidity.

Maintenance

Maintain the massage table regularly. Make sure that all screws are tight. Also make sure that there are no cracks or splits in the wood.

Care instructions I Cleaning recommendation

Danger of damage due to improper care of leatherette and surfaces!

Disinfect the contact surfaces after each treatment with RUCK surface disinfection wipes, alcohol-free (item no. 2967002, item no. 2967102 or item no. 2967202) for medical devices. Tip: Using disposable covers helps to avoid direct skin contact (body grease and sweat are sometimes quite aggressive) with the leatherette and thus to protect it. Clean the leatherette surfaces with a damp cloth and clean with RUCK leatherette cleaner and care product (item no. 2947001). Please observe the exposure times of the disinfectants according to the product labelling. One minute for bacteria, five minutes for fungi. Further information on the disinfectant.

Do not use any concentrated disinfectants, solvents, cleaning agents or liquids that contain alcohol (>50 % alcohol content). Do not use spray disinfection.

Take great care to ensure that the cleaning utensils used are clean. If mineral contaminants such as grains of sand or dust are embedded in the cleaning agents, this can destroy the surface. Do not use any scouring agents, such as abrasive cleaners, cleaning sponges or similar items that may contain abrasive particles. Residues of leaked fluids must be removed from the surface immediately with a soft cloth.

For gentle cleaning of the surfaces, use a soft cotton cloth or the RUCK microfibre cloth, which is also suitable for wood surfaces (item no. 2961601).

Caution: Other cleaning cloths with plastic fibres can have an abrasive effect and consequently they can damage the surface.

Over time, traces of use can appear on the surfaces. These are more or less noticeable depending on light incidence. They do not constitute a lessening of utility. As is the case with all surfaces over the long term, due to the wear involved in the daily routine of the medical practice, they cannot be avoided. Spots or stains caused by liquids that may corrode the surface (such as propolis, tea tree oil, cleaners that contain chlorine or similar substances) are categorically excluded from the warranty.

Risk of disease transmission

Insufficient disinfection can lead to transmission of disease. Always comply with the requirements of the responsible national body for hygiene and disinfection. Disinfect the product on a regular basis (see instructions for disinfection and care).

Differences in the wood grain pattern

Your massage table is made of selected hardwoods and has a lacquer finish that requires no special care. If desired, you can polish the wood with a standard wood care product. Store the massage table in a dry place and avoid wet or damp environments. Should the massage table still get wet, dry it thoroughly with a dry cloth.

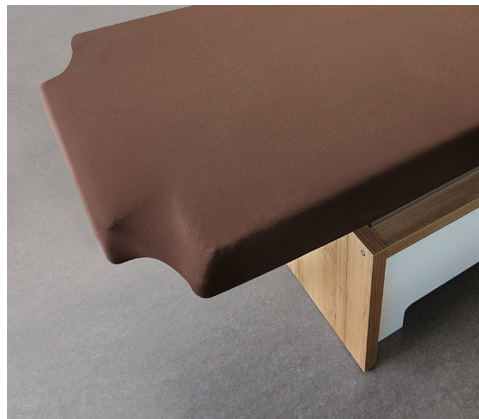
Cushion discolouration due to articles of clothing

Clothing, such as jeans, can cause discolouration of the cushion surfaces. If possible, you should attempt to remove discolouration residues with a soft cloth and the RUCK® leatherette cleaner and care product, 500 ml (item no. 2947001). RUCK recommends that you use disposable covers whenever possible. After cleaning and disinfection, surface disinfectants must be allowed to effectively evaporate before the next treatment; i.e. allow surface disinfectants to thoroughly dry as specified by the manufacturer, but do not wipe surfaces dry.

We recommend the following:

After each treatment, change the massage table cover (e.g.: Stretch cover, art. no.: 3024301).

Accessories

Even more matching accessories for your massage table:

3221501

Half-roll for the RUCK® mobile massage table, brown

3221301

Towel cloth cover for RUCK® mobile massage table, white

4070101

RUCK® beautykultur Disposable Face Rest Cover, white

4072501

RUCK® beautykultur treatment chair pad, white, waterproof and oilproof

Spare parts

If you need spare parts for the massage table, please contact RUCK SERVICE:
Phone +49 7082 - 944 22 55 | fax +49 7082 - 944 22 59 | e-mail service@hellmut-ruck.de

Warranty

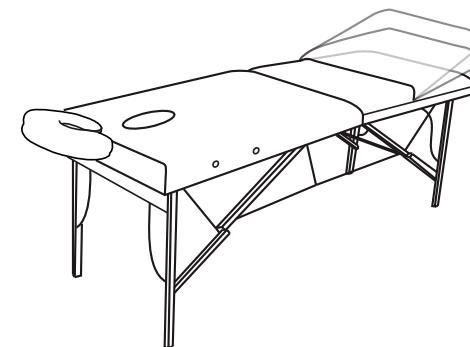
The product comes with a 2-year warranty. Normal wear and tear of surfaces, handles, clamping devices and/or moving components such as rollers, bearings, plug connections, suction hoses, and so on, are excluded from this warranty. Traces of use and signs of wear and tear due to age, service life and operation which are based on normal use corresponding to the performance specifications and/or the nature of the product are in accordance with the contract and are not defects from a legal standpoint.

The warranty is valid from the date of purchase when the product was first bought. The date of purchase must be confirmed by the sales receipt. Within the scope of the warranty, HELLMUT RUCK GmbH will, at no charge, either repair or replace the defective products, at RUCK's discretion. This requires carriage-paid shipment of the defective product together with the purchase receipt. The local agent is responsible for claims made outside of Germany. In case of doubt, please check with RUCK before returning the device.

The warranty becomes null and void if we find that the problem was caused by improper or incorrect installation, assembly or usage, by non-observance of the instructions for use, by external intervention or as a result of an unauthorised repair or modification. The warranty is limited to repair or replacement of the product. Further liability (loss of revenue or earnings in particular) is excluded. However, if it is determined that it is a problem that is not covered by the terms of the warranty or that the period of the guarantee has elapsed, any costs for inspection and repair must be borne by the customer. We reserve the right to make changes to and deviations from the technical design.

Technical specifications

Lying surface (L/W): 186/70 cm
Weight: 21 kg
Load capacity: max. 200 kg
Height: adjustable from 64 to 86 cm
Headrest: incl. breathing hole
Frame: Beech wood
Upper material: high quality PVC, oil and water resistant
Upholstery: Fine cellular foam (height 6 cm)
Armrest: adjustable to two heights

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HELLMUT RUCK LTD