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#### EN | Replacement of the joint with maintenance set 4911101.

Loosen the screw. Remove the nut and discard it (1). Remove the joint shells and dispose of them. Spray the inside of the new joint shells with maintenance spray. Place the upper joint half (with recess for the screw head) in the joint bushings. Spray the joint bushings with maintenance spray (2). Insert the lower joint half together with the new nut (3). Insert the screw and tighten it (4).

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**EN |** Open the Velcro fastener and pull out the telescopic rods until the three red locking buttons (B) are in the locked position. Unfold the seat.

**WARNING! Risk of injury from loose or damaged components!**

Before each use: Check the condition of all components and connecting elements. Pay particular attention that the seat is firmly connected to the bars (A). Only sit on the chair if all parts are intact and firmly connected to each other.

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**EN | Replacement of the 2-part joint every 24 months**

The joint is a wearing part and must therefore be replaced every 24 months from the date of purchase.

**WARNING! Risk of injury due to breaking joint.**

If the joint is not replaced, it can break under load and injure people.

Send in the product and we will take care of the exchange.

You can also replace the joint yourself in the comfort of your own home. For this purpose, order the maintenance set (item no. 4911101). The set contains 2 joint halves, a locking nut, maintenance spray and a Torx key. You will find the replacement instructions on the back.

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