# RUCK® Seal Check-Teststreifen

## RUCK® Seal Check Test Strips

Art.-Nr.: 2644401, 100 Stück pro Packung | Item no.: 2644401, 100 pieces per pack

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Seal Check test strips for manufacturer-independent testing of your sealing device

1. Directions for use

- 1. Switch on the sealing device and wait until the set sealing temperature is reached.
- 2. Insert the Seal Check test strip into a 20 cm wide foil, between the foil and the paper side, with the black indicator side visible through the foil.
- 3. When carrying out the sealing process, the sealed seam must be guided over the black indicator side.

2. Evaluation features

- 1. Continuous sealed seam width of at least 6 mm over the entire width of the film.
- 2. No perforated or torn areas.
- 3. No detached or unravelled sealed seam material.
- 4. No seam gaps or open seal seams.

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4. Auswertung

3. Implementation

The sealing device must be ready for use and have reached the necessary working temperature (between 170°C and 180 °C). The inspection is carried out using a standard clear packaging, 180 mm wide or larger, to demonstrate a full rotation of the sealing wheel. Place the test strip parallel to one edge of the transparent packaging. The black printing area must be arranged lengthwise to the sealed seam. Sealing should conform to the standardised instructions for clear packaging. The Seal Check verifies the quality of adhesion to the paper, through the contrast on the special print pattern. With the correct fusion of the inner film layer and its correct pressure on the paper, an uninterrupted sealed seam is created. Insufficient sealing temperatures also become visible.

The manufacturer recommends carrying out the test once a week in the field of podiatry.

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- 1. Pressure problems
- 2. Sealing temperature not constant
- 3. Correct sealed seam

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HELLMUT RUCK GmbH